



STERILIZER CARE

Daily

- A. Wash exterior of sterilizer according to your facility's procedure for clinical contact surfaces. Use only quaternary disinfectants to disinfect unit. See User Guide for more information.
- B. Examine door gaskets for damage. Clean gaskets and mating surfaces with a damp cloth.

Weekly

Clean Internal Surfaces

- A. Drain water from the reservoir using drain tube on front of unit.
- B. Remove trays, rack, and plate. Wash removed items and inside chamber with mild soap or Speed-Clean and water.
- C. Refill reservoir with clean water.

Monthly

- A. With a cooled chamber, drain reservoir and fill with clean water. Add one ounce of Speed-Clean sterilizer cleaner directly to the bottom of chamber.
- B. Run one Pouches cycle.
- C. Press Stop button when Dry Cycle begins.
- D. Drain reservoir and refill with clean water.
- E. Rinse by running one Unwrapped cycle. Press the "Stop" button when the drying cycle begins.
- F. Drain and refill reservoir with clean water, then allow sterilizer to cool.